Compact Rollator

Able 2

Assembly Instructions

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Instructions de montage

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Note: Cable needs to be placed outside seating area and frames

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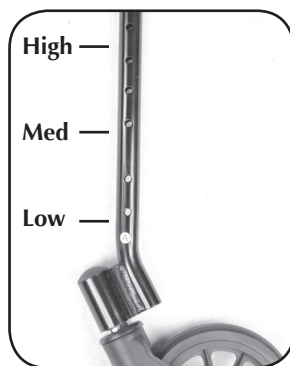
COMPACT WALKERS

Instructions for Use, Cleaning and Care

These instructions are guidelines only. Use only as instructed by your healthcare provider

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COMPACT WALKERS (PR30282/BL, PR30275/BL, PR30275/SL)

Weight capacity of compact walker is 114kg.

WHAT ARE COMPACT WALKERS USED FOR?
A walker or "rollator" is a light weight frame that is used to provide walking support, with the added advantage of swivel front wheels and seat (Rollator only)

How Do I Adjust The Roller Walker?

1. Stand with your shoulders relaxed and your arms hanging loosely at your sides.
2. The handlebar height should be at the crease of your wrist when your arms are relaxed at your sides.
3. Loosen knobs on the outside of the handlebars, remove the bolts, then move the handlebars to the desired height.
4. Re-install the bolts and screw the knobs to the bolt. Ensure the bolt head is properly aligned with the hole before tightening fully.
5. You should be able to sit on the seat with the balls of your feet touching the ground (Rollator only).

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1. Lean on the walker, allowing it to roll forward with control as you step forward.
2. To slow the walker down (Rollator only), squeeze or pull on the handlebar.
3. Before using the seat activate the parking brake by pushing down on the handlebar.
4. To release the brake, carefully squeeze the brake handles again, being careful not to pinch your fingers.

⚠ WARNING Do not use the walker on stairs or escalators. Pay special attention on ramps or slopes.

COMPACT WALKER CARE AND MAINTENANCE

- Clean the walker with mild soapy water.
- Periodically check the wheels and make sure wheels spin and swivel freely, and are not excessively worn. Clean or replace if necessary.
- Periodically check the brakes to assure locking mechanism is working properly and holds when locked.
- Store walker indoors. Cover to prevent dust buildup if not in use for a long period of time.

These instructions are guidelines only.

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Able2 UK Ltd