Importer:

ABLE2 EUROPE B.V.
Koperslagerij 15
4762 AR Zevenbergen
The Netherlands
WWW.ABLE2.NL

ABLE2 UK LTD.
WWW.ABLE2.UK

ABLE2 IBERIA S.L.
WWW.ABLE2.ES

**ES**

Soporte apoyabrazos abatible de pared

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FOLD AWAY GRAB RAIL

FOR YOUR SAFETY AND COMFORT

Valentine International Ltd. Taiwan
8th Floor, No. 149, Sec 2, Ta Tung Road
221 Hsichih District, New Taipei City
Taiwan R.O.C

UK

REP

UK Aplan Corporation Ltd.
Wandle House Associates, 47,
Wandle House, Wandle Road,
CROYDON, Surrey, CR0 1DF,
United Kingdom

EC

REP

KOMAS Medical Technology GmbH
Sternstraße, 67
40479 Düsseldorf, Germany.

a brand of
Able2

EN

Fold away grab rail

! Please inspect all parts carefully for cracks/breaks prior to use.

Usage:

The fold away grab rail is intended for use as an aid in support while sitting down or standing up. The rail can be folded up against the wall when not in use. There is a small tube within the rail to hang toilet tissue.

Caution:

Before use, make sure that the rail is installed securely. This product is intended as an aid in sitting and standing. It is not intended for full weight bearing. DO NOT SIT ON THE RAIL.

Maximum load weight 70 kg.

Mounting:

Preferably have the installation of the Fold away grab rail done by qualified electricians or mechanics. The wall must be capable of supporting the weight of the rail and the load imposed on it by the user. Determine the type of wall the rail will be mounted on, and use the correct screws and plugs (not included)!

Maintenance:

Clean with mild detergent and water; dry thoroughly.

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